

# ProDetect® Drug of Abuse Test (MOP)

⊕ High sensitivity, specificity and accuracy. ✓ MDA registered.




Manufactured by:  
**MEDIVEN®**  
MEDICAL INNOVATION VENTURES

Medical Innovation Ventures Sdn. Bhd. (MEDIVEN®)  
1st Floor, Plot 88F, Lintang Bayan Lepas 10, Bayan Lepas Industrial Park, Phase 4,  
11900 Bayan Lepas, Pulau Pinang, Malaysia.  
Email: [sales@mediven.com.co](mailto:sales@mediven.com.co) | [export@mediven.com.co](mailto:export@mediven.com.co)  
Office / Fax: +604 305 2730  
Website: [www.mediven.com.my](http://www.mediven.com.my)

## ABOUT OUR TEST KIT

A rapid chromatographic immunoassay for the detection of morphine in human urine at a cut-off concentration of 300 ng/mL. This test also detects other morphine-related compounds as shown in the table below. This assay provides only a qualitative, preliminary analytical test result. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are used. For *in vitro* diagnostic and professional use only.

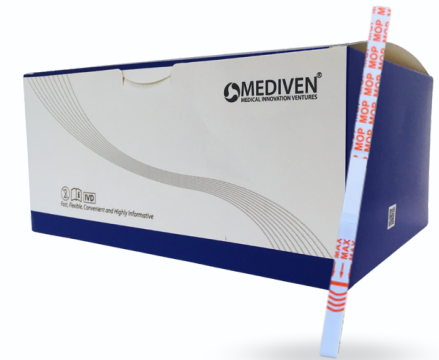

Tests and cut-off concentrations:

Compound	Concentration (ng/mL)	Compound	Concentration (ng/mL)
Codeine	200	Morphine	300
Ethylmorphine	6,000	Norcodeine	6,000
Hydrocodone	50,000	Normorphone	50,000
Hydromorphone	3,000	Oxycodone	30,000
Levorphanol	1,500	Oxymorphone	50,000
6-Monoacetylmorphine	300	Procaine	15,000
Morphine 3-β-D-glucuronide	800	Thebaine	6,000



**FAST**  
Test results in 5 minutes.



**USER-FRIENDLY**  
Simple test procedure.



**RELIABLE**  
High sensitivity, specificity, and accuracy.



**NON-INVASIVE**  
Uses a urine sample for testing.

## PERFORMANCE CHARACTERISTICS

Diagnostic sensitivity and specificity

Percentage agreement with GC/MS (%)	
Positive agreement	95.0
Negative agreement	95.3
Overall agreement	95.2

## ORDERING INFORMATION

Catalog No.	Product Name	No. of Tests/Kit
PR-DMO	ProDetect® Drug of Abuse Test (MOP)	50