

ProDetect® Drug of Abuse Test (OPI)

⊕ High sensitivity, specificity and accuracy. ✓ MDA registered.



Manufactured by:




Medical Innovation Ventures Sdn. Bhd. (MEDIVEN®)
1st Floor, Plot 88F, Lintang Bayan Lepas 10, Bayan Lepas Industrial Park, Phase 4,
11900 Bayan Lepas, Pulau Pinang, Malaysia.
Email: sales@mediven.com.co | export@mediven.com.co
Office / Fax: +604 305 2730
Website: www.mediven.com.my



ABOUT OUR TEST KIT

A rapid chromatographic immunoassay for the detection of morphine in human urine at a cut-off concentration of 2,000 ng/mL. This test also detects other morphine-related compounds as shown in the table below. This assay provides only a qualitative, preliminary analytical test result. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are used. For *in vitro* diagnostic and professional use only.

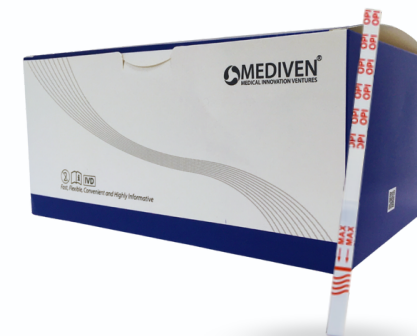

Tests and cut-off concentrations:

Compound	Concentration (ng/mL)	Compound	Concentration (ng/mL)
Codeine	2,000	Morphine	2,000
Ethylmorphine	3,000	Norcodeine	25,000
Hydrocodone	50,000	Normorphone	50,000
Hydromorphone	15,000	Oxycodone	25,000
Levorphanol	25,000	Oxymorphone	25,000
6-Monoacetylmorphine	3,000	Procaine	50,000
Morphine 3-β-D-glucuronide	2,000	Thebaine	25,000



FAST
Test results in 5 minutes.



USER-FRIENDLY
Simple test procedure.



RELIABLE
High sensitivity, specificity, and accuracy.



NON-INVASIVE
Uses a urine sample for testing.

PERFORMANCE CHARACTERISTICS

Diagnostic sensitivity and specificity

Percentage agreement with GC/MS (%)	
Positive agreement	96.7
Negative agreement	93.8
Overall agreement	95.2

ORDERING INFORMATION

Catalog No.	Product Name	No. of Tests/Kit
PR-DOP	ProDetect® Drug of Abuse Test (OPI)	50